MIKROBIOOMI UURING

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MICROBIOME TEST

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What is a microbiome test?

A microbiome test is the study of microbial communities in the gut which helps to make nutrition related decisions. For example, the results show whether your gut's microbial communities are varied enough, which bacteria are found in your gut, whether you consume enough fibre, fat, protein, etc.

Interpretation of these results requires the help of a nutritionist.

The test is based on a **stool sample**, a **3-day nutrition diary** kept before taking the stool sample, and your responses to **the health questionnaire**. If any of these three elements is missing, the results cannot be issued (a more precise description is available in the Patient Portal under the microbiome test terms and conditions).

The test is carried out by SYNLAB Eesti in cooperation with Elsavie.

6 steps to the result

1. After placing an order/making the payment, the sample collection kit is sent to you by SmartPOST or you can alternatively collect it from the nearest SYNLAB facility.

2. Keep the nutrition diary for 3 days; you can view the template of the diary beforehand at MINU.SYNLAB.EE/MIKROBIOOM.

3. Collect the stool sample on the fourth day.

4. On the day of collecting the stool sample enter the nutrition diary and fill in the health questionnaire at MINU.SYNLAB.EE/MIKROBIOOM.

5. Bring the sample to SYNLAB or send it by post within 7 days.

6. Your results will become available in 4 weeks at MINU.SYNLAB.EE/MIKROBIOOM/KYSIMUSTIK.

Instructions for collecting the sample

The sample collection kit contains:

1. A faeces catcher which can be attached onto the toilet
2. A sterile cotton swab
3. A sample container with a solution
4. Instructions for use
5. User ID and password (ID and PIN) on the reverse side of the instructions
6. A safety envelope (paid)

How to collect a sample?

1. Read the instructions for use

2. Collect data to fill in the questionnaire

Immediately after taking the sample fill in the questionnaire, in order to allow for more precise interpretation of the results which is why we recommend reading it through beforehand. The questionnaire contains questions about your health, consumption of food supplements, and food consumed in the last 3 days before collecting the sample. The questionnaire is available at MINU.SYNLAB.EE/MIKROBIOOM. You can find your personal user ID and password (ID and PIN) for logging in on the reverse side of the instructions.

3. Collect the stool onto a faeces catcher

Please wash your hands. Attach the faeces catcher onto the toilet as shown on the figure. While attaching the catcher onto the toilet, make sure that the catcher does not come into contact with toilet flush water, cleaning products or urine, as it disintegrates when it gets wet. Collect the stool onto the faeces catcher.

4. Collect the faeces onto the cotton swab

Carefully open the sterile cotton swab wrapper, being careful not to touch the cotton part of

the swab with your hands. Stick the sterile cotton swab into the middle part of the faeces (approximately 1 cm depth), slightly swirl it around to make some faeces stick onto the swab. Repeat this action 2 more times in different parts of the stool portion, so that the swab has no more than a pea-sized amount of faeces on it.

5. Place the sample into the cup

Open the sample container packaging, turn the cap on the liquid filled container and open it. Do not dispose of the liquid in the container or replace it with any other liquid. Turn the cotton swab covered with the sample material inside the solution, to allow the cotton to get wet. Important: Only a small amount of sample material is required for the test; the solution and sample mixture in the tube must be in liquid form. When the mixture contains too much faecal matter, it is not possible to analyse the sample. Then bend and break the swab from the point marked on the shaft, leaving the cotton tip inside the sample container. Throw the other end of the swab into the wastebin. Close the cap carefully.

6. Finish collecting the sample

Flush the faeces catcher down the toilet. Wash your hands once again. Write the date, month and year of collecting the sample onto the sample container.

7. Fill in the questionnaire

On the day of collecting the sample, be sure to fill in the 3-day detailed nutrition diary and questionnaire at

MINU.SYNLAB.EE/MIKROBIOOM. Your personal user ID and password (ID and PIN) for logging in can be found on the reverse side of the instructions.

8. Send or bring the sample container to SYNLAB

Place the sample container in the security envelope and bring it to SYNLAB or post it to an Omniva parcel machine at your earliest convenience, but no later than in 7 days after collecting the sample. The kit includes an envelope for returning the sample, which has been paid for you.

9. Keep your ID and PIN

Your personal user ID and password (ID and PIN) allow you to view the results at MINU.SYNLAB.EE.

What happens next?

SYNLAB sequences the bacterial DNA from your stool sample and performs test in cooperation with the Center of Food and Fermentation Technologies laboratory in Tallinn, using a method developed by them.

You will be able to access your microbiome test results after 4 weeks through the SYNLAB Patient Portal MINU.SYNLAB.EE (you can order the system to notify you when the result arrives).

If you need help with interpreting the test results and wish to get nutrition recommendations, you will find the contact details of qualified nutritionists from the Patient Portal under the microbiome test section. This is a paid service and the price is coordinated by each nutritionist.

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